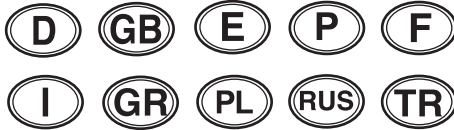
Stilglock

# NUK Elektrische Milchpumpe NUK Electric Breast Pump

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**Käufer:**

## Dear Mother,

Breast milk for babies is without a doubt the most important source of life. This is why NUK is supporting you with the NUK Breastfeeding Programme. With practical and convenient breastfeeding aids for your every need, we want you and your baby to enjoy the breastfeeding phase. This easy-to-handle breast pump is designed to help you and your baby when breastfeeding is

not possible or you wish to combine breast and bottle feeding.

Please take 5 minutes to carefully read these instructions before using your NUK Electric Breast Pump for the first time.

We wish you and your baby all the best.

## 1. Important Information before Using for the First Time

Clean and sterilise all parts (except pump unit **1** and mains adapter) before using the NUK Electric Breast Pump for the first time (see "9. Cleaning and Sterilising" on page 18).

## 2. Pump Description

### 2.1 Purpose

The NUK Electric Breast Pump is designed to help pump breast milk into a receptacle. The milk can be pumped directly into the NUK Breast Milk Container, where it can be stored, frozen and when required, heated up and fed to you baby.

### 2.2 Functional Description

The NUK Electric Breast Pump has an electric suction pump and offers the following benefits:

- Both mains and battery operation is possible (power cable supplied)
- Suction strength can be individually adjusted
- Simulates the natural sucking action when the interrupt button is activated
- Stand supplied for stability

The breast pump can be used together with any NUK standard bottle, NUK First Choice Bottle or the NUK Breast Milk Container.

### 2.3 Contraindications

If the breast is sore, enflamed or infected, the NUK Electric Breast Pump should preferably not be used or if it is, with extra special care. In each case you should consult your doctor prior to use.

### 2.4 Side Effects

If the recommended short pauses in operation are not adhered to, this may result in bruising of the nipple tissue. No potentially dangerous side effects have been observed from using the NUK Electric Breast Pump, however should you experience any unexpected reaction, please consult your midwife, health visitor, doctor or pharmacist.

### 3. Safety Information

For your own safety please observe the following:

#### 3.1 General Information

- Please read these instructions carefully before using the pump. They are an essential part of the NUK Electric Breast Pump and should always be to hand.
- The NUK Electric Breast Pump should only be used for its intended purpose (see "2.1 Purpose" on page 12).
- Check whether the necessary voltage on the mains adapter **15** (see nameplate) is compatible with the mains voltage.
- Never immerse pump unit **I** in water, as there is a danger of electrocution and the pump unit may become damaged if water penetrates.
- Never expose the device to direct sunlight, as the UV rays and overheating may make the plastic brittle.
- Always keep pump and batteries away from children, as the small parts can be swallowed which may cause choking.
- The batteries should be removed once they are dead or if the device is not going to be used for a while, as leaking batteries may damage the device. Dead batteries should not be disposed of with household waste but taken to a special collection point.
- Please note that any claim under guarantee or for liability will be invalid if accessories are used that are not recommended in these instructions or if original spare parts are not used for repairs. This also applies to repairs undertaken by non-qualified persons.
- If accessories from other manufacturers are used, the pump may not function correctly.
- In the event of a malfunction of the pump, repairs may only be undertaken by the manufacturer, otherwise the warranty will be invalid.

- If the breast milk is intended for a premature baby, it is essential that you follow your doctors advice.

#### 3.2 Pump Labelling

The following labels are to be found on pump unit **I**:

| Symbol | Meaning  |
|--------|--|
|        | Electrically protected device, Protection Class II |
|        | Electro medical device, Type BF                    |
|        | Observe operating instructions                     |
|        | D.C.   |

#### 3.3 Hygiene

- When handling breast milk, hygiene is of particular importance to ensure that your child develops healthily.
- Please observe Section "9. Cleaning and Sterilising" on page 18 to prevent infection or bacterial contamination and to prevent the device from becoming damaged.
- The pump unit should only be cleaned using cleaning tube **II**!
- **Only if the breast pump is carefully cleaned will it attain its full suction power.**
- Never open the casing of pump unit **I** and **never remove the red clamping ring from the pump unit, as this will make the warranty invalid.**

### **3.4 NUK FIRST CHOICE Bottle, NUK Breast Milk Container and NUK Standard Bottle**

- If the bottle is damaged from the inside i.e. by scratches, it should be replaced for hygiene reasons.
- All individual parts not used should always be kept out of the reach of children as there may be a danger of choking.
- Bottles and accessories may become discoloured by tea and fruit drinks.
- Feeding bottles which are sealed and contain feeds should not be put in the microwave as there may be a danger of explosion or material damage. Uneven heating may cause scalding to your child therefore please check the temperature before feeding.
- NUK bottles and teats are only compatible with the relevant system, for example NUK First Choice.

### **3.5 NUK Teats**

- Teats should be stored in a hygienic condition (clean and dry) at room temperature and away from light sources.
- For hygiene reasons, before using for the first time, teats should be sterilised in boiling water for 5 minutes.
- Teats should also be cleaned before each use.
- Teats should not be exposed to direct sunlight or heat, nor should they be left in sterilising solution beyond the period specified, as this may cause damage to the material.
- For hygiene and safety reasons, teats should be replaced every 1-2 months.

### **For the health and safety of your child: IMPORTANT!**

- This product may only be used under adult supervision.
- A teat should never be used as a soother.
- Sustained sucking of sweetened feeds, fruit juices or fruit acids over a long period may cause tooth decay.
- Always check the temperature before feeding.
- Check the teat each time for excessive use by pulling the teat in all directions – in particular where teething infants are concerned. Discard at the first signs of any weak points or damage to avoid the danger of small parts being swallowed.

## 4. Designation of Components

- |  |  |
|--|--|
| <ul style="list-style-type: none"> <li>1 Pump unit</li> <li>2 Battery compartment</li> <li>3 Connection for mains adapter</li> <li>4 On/Off switch</li> <li>5 Suction interrupt knob</li> <li>6 Suction strength regulator</li> <li>7 Sealing ring</li> <li>8 Connecting element<br/><b>(only for wide-necked bottles Art.-No. 10.252.059, 10.749.036)</b></li> <li>9 NUK First Choice Bottle PC 150 ml /<br/>NUK Breast Milk Container</li> </ul> | <ul style="list-style-type: none"> <li>10 Stand for NUK First Choice Bottle</li> <li>11 Cleaning tube</li> <li>12 NUK Teat Protective Cap</li> <li>13 NUK Teat</li> <li>14 NUK Screw Ring</li> <li>15 Mains adapter<br/><b>(not for Art.-No. 10.252.025)</b></li> <li>16 Stand</li> <li>17 NUK Standard Bottle PC 125 ml</li> <li>18 Nipple ring</li> <li>19 Suction part</li> </ul> |
|--|--|

## 5. Tips for Successful Operation

The correct pumping action needs practice. You should first become accustomed to your new breast pump.

### 5.1 Frequency of Pumping

The frequency of pumping will depend on individual requirements.

If you will be apart from your child and/or in order to sustain milk production, the natural feeding rhythm should be maintained. You should therefore pump your breast milk 6 to 8 times a day, including once during the night.

In all other cases you can use the pump to suit your own individual needs.

### 5.2 Application

Breasts should be pumped alternately.

You should alternate between the breasts when the milk flow slows down or ceases.

Pumping time will vary considerably depending on individual requirements.

The following intervals have been tried and tested but may vary according to the individual:

- Right 5 - 7 minutes; left 5 - 7 minutes
- Right 3 - 5 minutes; left 3 - 5 minutes
- Right 2 - 3 minutes; left 2 - 3 minutes

### 5.3 Milk Quantity

Please do not be surprised if initially it takes a while for the milk to start to flow. This is not unusual. In fact it is quite normal for the following reasons:

**Milk production will adapt itself to the daily requirement.**

This means that the quantity of milk produced may vary considerably.

- Always ensure that you drink plenty of fluids which also helps stimulate milk production.
- Do not set the suction pressure too high. A high suction level does not increase the quantity of milk produced.

### 5.4 Milk-producing Reflex

With breastfeeding, putting your baby to the breast stimulates the milk-producing reflex and the milk will begin to flow. You can also stimulate the milk producing reflex as follows:

- Find yourself a quiet place to sit, where you can completely relax when using the breast pump.
- You should assume a comfortable and relaxed posture.

- Before commencing, lay a warm compress on your breasts for a few minutes (e.g. the NUK Breast Compress) to help relax the breast tissue.
- Maintain eye-to-eye contact with your baby. If this is not possible, then look at a photo of your baby.
- The milk-producing reflex releases the flow of milk to both breasts simultaneously. Any surplus milk leaking from the breast not currently being pumped can be collected using the NUK Breast Shell.

## 6. Assembling the Pump (Diagram ①)

1. Connect pump unit **1** to suction part **19**.
2. Place nipple ring **18** into the suction part.
3. Place sealing ring **7** inside the thread of the suction part.
4. **NUK Standard Bottle:**  
Screw NUK Standard Bottle **17** onto the suction part.

### NUK First Choice Bottle / NUK Breast Milk Container:

Firstly, screw connecting element **8** onto the suction part and then the NUK First Choice Bottle / NUK Breast Milk Container **9** to the connecting element. Ensure that the NUK bottle / container is screwed tightly to the connecting element. The electric breast pump will only function if a vacuum is able to build up.

## 7. Power Supply

For **Battery operation** 2 Type AA, 1.5V batteries are required.

1. Remove the red cover from Battery Compartment 2 and insert the batteries, ensuring that they are positioned correctly according to the symbols in the compartment! (Diagram ②).
2. Then replace the red cover, ensuring that it clicks into place. The device must not operate without the battery cover being in place.

**Rechargeable batteries** may limit the power of the pump.

For **mains operation** use only the enclosed adapter, Model AD-35-0350, or for the UK, Ireland and Cyprus: Model AD-35-0350BS.

1. Connect the hollow plug of mains adapter **15** to connection **3** of the pump unit and insert the adapter into a socket (Diagram ).
2. Check the voltage before operating the pump. The voltage specification on the device must be the same as the mains voltage. In the event of damage to the adapter a new one can be obtained from Mapa GmbH or from one of our distributors abroad.

## 8. Operation (Diagram ④)

**N.B.!** As hygiene is of great importance, you should always wash your hands before using the pump and observe the advice in "10. Tips on Hygiene" on page 19.

### Important

L = low suction  
H = high suction

First set suction strength regulator **6** to a minimum by rotating the external ring to the left towards "L".

1. Place suction part **19** on your breast so that the nipple is positioned in the centre. Gently press on the suction part in order to ensure a good seal when pumping the milk. **Always hold the pump in a vertical position.**

**N.B.!** If the pump is tilted too much, milk will be sucked into the pump, resulting in possible damage to the device and potential hygiene problems.

2. Switch on the pump. Do this by turning red switch **4** upwards towards "ON".
3. Adjust the suction power using rotary knob **6** until the optimum suction strength is attained.

The suction strength during pump delivery will be on the lowest setting (L).

If the suction strength is too weak, rotate to the right towards "H". If this is too strong, rotate the knob to the left towards "L".

4. Pump operation can be interrupted by pushing and releasing knob **5**. Pushing and releasing the knob during operation simulates the natural sucking process.

**N.B.!** These short pauses in suction are also important to help prevent the breast tissue from bruising.

5. When you wish to stop pumping push knob **5** to gently release the pump from the breast.

6. To switch off the breast pump move switch **4** to the "OFF" position.
7. After use, place the breast pump onto stand **10** or **16**, for stability.

### 8. NUK Standard Bottle:

Unscrew the suction part and pump unit from NUK Standard Feeding Bottle **17**.

### NUK First Choice Bottle/

### NUK Breast Milk Container:

Unscrew the suction part and pump unit and then the connecting element from the bottle/container.

You can now feed your baby the milk by screwing teat **13** and screw ring **14** to the bottle.

To store the milk, secure the bottle using the protective cap provided, which you attach with screw ring **14** to the bottle. Please refer to Section "11. Storing the Milk" on page 19.

## 8.1 NUK Breast Milk Container

**Important:** When freezing breast milk do not fill the container above the 150 ml/5 fl. oz. level, as the milk will expand during freezing and therefore the container may burst.

1. Unscrew the pump unit from the NUK Breast Milk Container.
2. Secure the NUK Breast Milk Container using the screw ring supplied and the associated sealing disc.
3. Using a water-soluble pen, write the date and time of storage in the specified field on the container.

## 9. Cleaning and Sterilising

Hygiene is of priority when handling breast milk and all parts that come into contact with it. The pump should therefore only be used in a clean and

sterile condition.

### 9.1 Intervals

**After each use:**

- All **accessories** with the exception of pump unit **I** and power adapter **I5** must be thoroughly cleaned with water and sterilised **immediately after each use**.
- Pump unit **I** must be cleaned after each use to prevent sticking, especially if the pump has sucked in any milk.

### 9.2 Cleaning

1. **N.B.!** It is essential that the device is unplugged before cleaning.
2. Remove pump unit **I** from the suction part by rotating slightly.
3. Unscrew the suction part and with the NUK First Choice Bottle/NUK Breast Milk Container also the connecting element from the bottle.
4. All **accessories** with the exception of pump unit **I** and the mains adapter should be cleaned and rinsed under running water. Milk residues may result in the build up of bacteria and may cause damage to the material.

**Important:** The components should never be put in the dishwasher as detergents may damage the material.

5. Wipe pump unit **I** with a damp cloth. Never immerse the device in water.
6. Place cleaning tube **II** with the wide opening over the suction opening of the pump unit.
7. Fill a plastic bowl with warm water and immerse the tube (Diagram **5**)

8. Switch on the pump and let the water flow through the pump for a minimum of 30 seconds. To do this you may have to briefly reconnect the power supply.
9. Remove the tube from the water and suck in air.
10. Switch off the pump.
11. Pull out the cleaning tube.

### 9.3 Sterilising

You should follow the instructions on cleaning as specified in Section "9.2 Cleaning", each time before sterilising.

**Important!** Pump unit **I** should never be sterilised, as this would damage the electrics.

Only the **accessories** such as bottle **9**, connecting element **8**, teat **13**, suction part **19**, sealing rings **7**, nipple ring **18**, screw ring **14** and teat protective cap **12** may be sterilised.

- Use a NUK Steam Steriliser for this purpose, observing the relevant operating instructions, or
- Sterilise the components in boiling water for at least 5 minutes.

**N.B.!** On no account should items be sterilised in the microwave, as this can result in damage to the material.

**Important!** Ensure that the accessories have completely cooled before using again.

## 10. Tips on Hygiene

Hygiene is particularly important when using the breast pump. This applies both to all individual pump components as well as to personal hygiene. Showering or washing the nipples with water is

recommended on a daily basis. Soap should preferably not be used, as this can cause irritation to the nipples.

## 11. Storing the Milk

If the breast milk is not to be used immediately, the bottle must be transferred to a suitable container, sealed and stored in the refrigerator.

**N.B.!** Only the bottle with lid should be placed in the refrigerator, not the complete NUK Electric Breast Pump.

### 11.1 NUK Standard Bottle

If the breast milk is not to be used immediately, the bottle should be sealed and stored in the refrigerator.

If the breast milk needs to be kept for longer than 72 hours, it should be frozen immediately in a suitable, clean, hygienic container. The NUK FIRST CHOICE and Standard Bottles are however, not suitable for this purpose.

#### Tip:

Use the NUK Breast Milk Container for freezing breast milk. To thaw out the container, place in the refrigerator or, if required immediately, stand in warm water.

### 11.2 NUK Breast Milk Container

The breast milk can be cooled down, frozen and thawed out again in the Breast Milk Container. The container can also be used for re-heating and feeding. Please always observe the advice below on handling breast milk.

### 11.3 Tips for Handling Breast Milk

- Breast milk can be stored for up to 72 hours at +4 °C in the refrigerator (but not in the refrigerator door).
- Maximum storage times are 2 weeks in the freezer compartment of the refrigerator or 6 months in the freezer at -8 °C.
- The milk may be thawed out in the bottle, either in the refrigerator or if required immediately, by standing in warm water.
- In order to attain the correct drinking temperature of approx. 37 °C, the container should be placed in the NUK Baby Food Warmer.
- **N.B.:** Breast milk must not be thawed out in the microwave, as this will destroy valuable nutrients as well as causing possible damage to the material of the bottle and may cause a risk of explosion. Uneven heating may result in scalding! The milk should therefore be shaken well before feeding to ensure that milk and milk fats are well mixed and that the warm milk is evenly distributed.
- Milk which has thawed out and then heated but not fed to your infant must not be refrozen but should be disposed of.
- More precise instructions as to long-term storage can be obtained from your midwife, health visitor, doctor or pharmacist.

## 12. Technical Data

|  |  |
|--|--|
| Medicinal product to 93/42/EEC   | Class I  |
| Temperature range<br>– Operation<br>– Storage                            | 0 °C to +40 °C<br>0 °C to +40 °C   |
| Power supply:<br>– Mains operation via power plug<br>– Battery operation | Input 230 V, 50 Hz, output 3 VDC, 500 mA,<br>polarity +/-, 5 x 2.1 mm hollow plug<br>requires 2 Type AA, 1.5 V batteries |

### EMC Guideline to EN 60601-1-2:2001:

| Guidelines and manufacturers declaration - Electromagnetic emissions   |            |  |
|--|------------|--|
| The NUK BP3 Electric Breast Pump is designed for use in the environment specified below. The customer or user of the NUK BP3 Electric Breast Pump should ensure that it is used in that type of environment. |            |  |
| Measurement of Interference Emission   | Compliance | Electromagnetic Environmental Guidelines   |
| HF-Emission of Interference to CISPR 14  | Complies   | The BP3 NUK Electric Breast Pump is not suitable for connecting to other devices.  |
| Emission of Harmonic Oscillations to IEC 61000-3-2   | Class A    | The BP3 NUK Electric Breast Pump is suitable for use in all fittings, including those in a domestic environment as well as those connected directly to a public supply network that also supplies buildings used for residential purposes. |
| Emission of Voltage Fluctuations/ Flickers to IEC 61000-3-3  | Complies   |  |

## 13. Disposal

In the European Union the symbol of a crossed-through waste bin on wheels ("wheelie bin") means that the product must be disposed of at a separate collection point. This also applies to the product and all accessories marked with this symbol. Products labelled as such must not be disposed of with normal household waste, but should be taken to a collection point for recycling electrical

and electronic equipment. Recycling helps to reduce the consumption of raw materials, thus protecting the environment.

Batteries must not be disposed of with household waste. It is every consumer's legal responsibility to ensure that batteries are disposed of at a specially designated collection point.

## 14. Guarantee

Provided that the device is handled with care and used in accordance with these instructions, the manufacturer will guarantee the product for 24 months from the date of purchase.

We guarantee to repair the device free of charge provided that it is sent to us postage paid and provided that the claim is for either material or design or construction fault. Other claims will be handled in accordance with the law.

We cannot be responsible for damage resulting from improper use of the device. Wear of the cable or plug is excluded from the warranty. For a period of up to 24 months from the date of purchase we will rectify any faults recognised as genuine without charge.

In each case the device must be sent together with this completed warranty form, or failing this, the purchase receipt or other proof of date of purchase must be supplied.

Without proof of date of purchase, a charge will be made without further consultation and the items will be returned COD.

Please ensure that you complete the warranty form immediately after purchase and keep this, or the till receipt as proof of purchase in a safe place. Mislaidd warranty certificates cannot be replaced.

Please keep distributor's address, Art. No. and Lot No. in case of complaints.

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NUK is a registered trademark owned by MAPA GmbH