NUK[®]

Nasensauger
Nasal decongester

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Industriestr. 21–25
27404 Zeven
Hergestellt in Deutschland

GB Newell Brands UK Limited

Halifax Avenue, Fradley Park
Lichfield, Staffordshire,
WS13 8SS
United Kingdom

IRL Intrapharma

Consumer Health
Magna Business Park
Citywest Road
Dublin 24, Ireland
Made in Germany

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GB NASAL DECONGESTER

- **Intended use: Safety nozzle for hygienically removing congestion from your baby's nose.**

Easy and safe to use:

Squeeze the pump bulb and then carefully position the opening against one nostril without inserting the nozzle into baby's nose. While doing so, press gently on the other nostril, holding it closed with your finger. Now slowly release the pressure of your fingers on the bulb. This will remove the released congestion gently and hygienically and your baby's nose is clear again.

For the safety of your child:

- **Warning! Do not insert nozzle into the nasal passage as this may cause injury.**
- **Cleaning:** Nozzle is removable. Clean pump bulb and nozzle thoroughly after use with warm water and shake off any excess. Leave parts with the opening upwards to dry separately.
- **Warning:** Do not sterilise in a microwave oven or steam steriliser as the decongester may become damaged. For sterilising place in boiling water for 5 minutes, using sufficient water to avoid damage to the decongestor.

Please retain product information and article/LOT no. for future use.

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