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GB Important information before using for the first time

i Read the warnings and the instructions for use carefully.

- Retain the packaging and this pack insert.

1. Product description

1.1 Proper use

As a therapeutic tool, the NUK Therapeutic Trainer makes an important contribution to overcoming weak sucking and to normalising malfunctions of oral motor functions (such as breathing through the mouth, sucking malfunctions, chewing, swallowing, talking and facial expressions). In addition, use of the NUK Therapeutic Trainer can have a positive impact on muscle tone and on coordination of the intra-oral and peri-oral muscles. This permits effective treatment of even stubborn habits, such as grinding the teeth, biting down hard and compressing the tongue which are associated with pain and cracking in the jaw.

The NUK Therapeutic Trainer can equally well be used for applications in breathing, voice and speech therapy.

The NUK Therapeutic Trainer is suitable for schoolchildren and adults with weak sucking or malfunctions of their oral motor functions.

1.2 Models

- Size **S** for schoolchildren
- Size **L** for adults

1.3 Description of function

Its shape means that the Therapeutic Trainer has a positive impact on oral motor functions.

1.4 Contraindication

The Therapeutic Trainer must not be used if there is any possibility of anaphylactic shock.

The Therapeutic Trainer is not suitable for administering medication.

The Therapeutic Trainer must not be used in the patient’s mouth simultaneously with other medical devices (for exception, see "4. Use"), drugs or personal care products. The individual case should always be agreed with the responsible consultant.

1.5 Side effects

The Therapeutic Trainer is made using natural latex which can cause allergic reactions.

Report all serious incidents involving the product to the manufacturer and to your responsible national body.

1.6 Labelling

Symbol	Significance
50°C	Temperature limit +5 °C to +50 °C
15%	Humidity, limit during storage 15 % - 75 % rh
	Product contains latex
	Lot
	Use by
	Keep out of sunlight
	Medical device
	Recycle the packaging
	Declaration of conformity Complies with all requirements of the applicable EU directives
	Manufacturer

2. Safety information

2.1 Definition of terms

The following indicator words can be found in these instructions for use:

! Warning!

Ignoring this warning may cause serious damage to life and limb.

! Caution!

Ignoring this warning may cause minor personal injury and/or material damage.

2.2 General information

! Caution!

- It is essential to check that the Therapeutic Trainer is undamaged before using it each time. Before each use, check that it resists tearing by pulling along on the part which is sucked.

- If the Therapeutic Trainer is damaged, discard and replace it. We recommend replacing the Therapeutic Trainer regularly.

- The Therapeutic Trainer must only be used under professional supervision following prior instruction and introduction.

- The Therapeutic Trainer must not be put in a dishwasher - the detergent will damage the material.

- The Therapeutic Trainer must not be put in a microwave – risk of explosion/material damage.

- The Therapeutic Trainer must not come into contact with greases or oils, as these may restrict its serviceability.

2.3 Hygiene

! Warning!

To prevent risks to health:

- Clean the Therapeutic Trainer before each use and sterilise it (see "3. Cleaning, sterilising, storing").

3. Cleaning, sterilising, storing

3.1 Cleaning

Clean the Therapeutic Trainer in just water. If it is heavily soiled, use a mild detergent.

3.2 Sterilising

Boil in water:

Approx. 5 minutes is sufficient boiling time. Make sure that the water level in the pan is always sufficient during boiling to prevent the Therapeutic Trainer being damaged. The water must not boil dry. Squeeze any excess liquid out of the part which is sucked.

3.3 Storing

The Therapeutic Trainer should be stored in hygienic, dry and cool conditions away from light. Select storage locations with a temperature and humidity at or below those of the ambient atmosphere (see "1.6 Labelling").

When stored in proper conditions, the Therapeutic Trainer can be used up until the date printed on the individual pack.

4. Use

The type and duration of use will be determined by the responsible consultant.

4.1 Myotherapeutic training

For use in myotherapeutic training according to the PADOVAN method, the shield of the Therapeutic Trainer can be provided with a perforation through which a slim tube is passed in order to allow small

quantities of liquid to be drawn up during certain suck/swallow exercises.

4.2 Perforations in the shield

During treatment, the NUK Therapeutic Trainer can be used in conjunction with a slim tube for suck/swallow exercises. The perforation required for this can be made using ordinary HSS drills for metal or using a punch.

Only use the shaded area of the shield shown in the illustration for the perforation. Maintain a minimum distance of ≥ 5 mm from the edge. Remove the burr from the perforation, once made, to prevent injury to the user.

5. Help with problems

In the event of problems, contact the responsible consultant.

6. Disposal

If the Therapeutic Trainer is damaged or has visibly aged (sticky, cracked), dispose of it in domestic waste.

7. Warranty

The statutory terms of the country in which the product is sold apply.

7.1 Service Centre

Newell Brands UK Limited Halifax Avenue, Fradley Park Lichfield, Staffordshire, WS13 8SS

United Kingdom

www.nuk.co.uk

In case of complaints, please keep the packaging with address, art. no. and batch no.

7.2 Manufacturer

Mapa GmbH Industriestr. 21-25 27404 Zeven Germany www.nuk.com

8. Declaration of conformity

The NUK Therapeutic Trainer for applications in speech therapy and physiotherapy is a medical device in accordance with directive (EU) 2017/745.

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