GB Safety information

Before use, please read carefully these instructions.

- Use the product only for the intended purpose described. • The nipple shield is an item for personal use and to prevent cross-contamination, must not be used by more than one mother. • If used hygienically, the nipple shield can be used by the same user several times. Please disinfect or boil to sterilize at least once a day. • We recommend that you take advice from a doctor, midwife or breastfeeding counsellor. • Before using each time, please check the nipple shield for damage and replace it at the slightest sign of cracking or bite marks - parts could come away and choke your baby.
- Please store the nipple shield away from light, clean and dry. We recommend using the protective case for this. To store, place the clean and dry nipple shields in the lid of the case open side upwards and then close it. • The nipple shields and protective case are not toys and should be kept out of the reach of children.

Please keep address, article no. and batch no. for future reference.

Made in Germany.

Application:

- 1 Before attaching the nipple shield, wash your hands thoroughly and wipe the breast clean with a damp cloth. Do not use either soap or alcohol, as these can cause dry skin and cracked nipples. To improve the adhesion of the nipple shield to the skin, you can moisten the inside of the nipple shield with water or breast milk.
- 2 Fold the contact surfaces of the nipple shield outwards (towards the tip) and position the nipple shield centrally on the nipple.
- 3 The alignment of the nipple shield depends on the breastfeeding position you prefer. Please ensure that the indented part of the nipple shield points towards the baby's nose. This allows direct contact between the nose and the breast.
- 4 Then arrange the contact surfaces flat so that they are in comfortable contact with the skin.

After breastfeeding, please wipe the breast clean using a damp cloth to prevent drying milk residues and microbial growth.

In the event of problems in use, please contact your breastfeeding counsellor, midwife or doctor.

When selecting the right size of nipple shield, the physical features of mother and baby should be taken into account. The nipple shield should completely fill the interior space (M: 20mm; L: 24mm) without the nipple shield creasing. The baby's lips should completely surround the nipple shield and contact the mother's breast to ensure an optimum flow of milk.

Cleaning the nipple shield: Important!

- Please use only water of drinking quality for the cleaning process described.
- Please clean the nipple shield immediately after use to prevent drying milk residues and microbial growth.
- If used hygienically, the nipple shield can be used by the same user several times. Please disinfect or boil to sterilize at least once a day.

Before first use and immediately after every further use, wash the nipple shield thoroughly with water and mild washing-up liquid. We recommend NUK washing-up liquid. After cleaning, please rinse off the nipple shield in clean water and leave to dry in a hygienic environment (for example in our NUK storage box or on a clean, dry, lint-free cloth). Material damage cannot be completely ruled out if product is washed in a dishwasher. As a consequence, we do not recommend washing in a dishwasher.

You should also disinfect or boil the nipple shield once a day to sterilize it.

- We recommend the NUK Vaporiser for disinfecting/sterilising.
- When boiling to sterilize, ensure there is enough water to prevent damage. The nipple shield must be completely covered with water to prevent damage. Boil for 3 minutes to sterilize. If boiling or steam-sterilizing with other products, discolouration is possible. This does not affect the function.

If the nipple shield is not being used, please store it dry and keep it in the enclosed storage box to prevent contamination. Please also clean the NUK storage box regularly. To do so, use NUK washing-up liquid and a clean, dry lint-free cloth.

MAPA Spontex UK Ltd., Berkeley Business Park, Wainwright Road, Worcester WR4 9ZS, Great Britain

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