D Liebe Mutter,

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(GB) Dear Mother,

Breast milk is the best food for a baby during the first six months of its life. To enable you and your baby to enjoy every moment of breastfeeding, NUK has developed a range of practical and comfortable breastfeeding aids for the systematic support of breastfeeding. The NUK Soft & Easy manual breast pump is designed to help you and your baby to enjoy every moment of breastfeeding as well as to express small quantities of milk. This pump is particularly suitable for expressing small quantities of milk and a brief expression before breastfeeding - this is because relaxed breasts make it easier for baby to nurse. In addition, the NUK Soft & Easy breast pump uses a vacuum to give inverted nipples their proper shape.

2.2. Description of function

The NUK Soft & Easy breast pump has the following features:
• Soft, contoured silicone cushion for a pleasant massage and a gentle expression
• Fewer parts for easy cleaning and use
• Wide base for stability
• Enables smooth expression
• To use the cylinder directly as a baby bottle (see fig. e), place a NUK Classic teat on the pump and secure it with a NUK Classic adapter ring. The teat and adapter ring are sold separately.

2.3. Contraindications

In the event of certain breast problems (infections, open cuts), the NUK Soft & Easy breast pump should only be used after consulting the doctor supervising treatment or your nursing consultant.

2.4. Safety instructions

- For your own safety please observe the following:
• Please read these operating instructions carefully before using the breast pump. The instructions are part of the NUK Soft & Easy breast pump and should be on hand at all times.
• If the breast milk is intended for a premature baby, it is essential that you follow your doctor's advice.
• The breast pump must be cleaned and sterilised before each use.
• Please observe point 4. Cleaning and Sterilisation to prevent microbial contamination and damage to the breast pump.
• If the pump components are damaged on the inside (e.g. by scratches), for hygiene reasons the components should be replaced.

2.5. When cylinder is used as a feeding bottle

For your child's safety and health:

WARNING!

- Always use this product with adult supervision.
• Never use feeding teats as a soothe. Continuous and prolonged sucking of fluids will cause tooth decay.
• Do not dip in medicine!
• Check the teat before each use by pulling it in all directions - especially when children are teething. If you notice even tiny cracks, teeth marks or visible ageing (sticky, swollen, cracked), change the teat as parts could come loose and be swallowed. Do not microwave.
• Use only original NUK parts.
• NUK product complies with EN 14350 "Child use and care articles - Drinking equipment".

3. Use

- 1. Use the NUK Soft & Easy breast pump only after cleaning it (see 4. Cleaning and Disinfection (Abb. d)).
- 2. Always wash your hands thoroughly before using the pump.
- 3. Push the silicone ring onto the piston (fig. b). To do this, place the silicone ring on the opening of the piston, and use the palm of your hand to push the silicone ring completely onto the piston. The ring has to completely fit in the recess at the lower end of the piston. Do not allow it to protrude at all. In particular, make sure that the silicone ring is not folded onto itself, which would prevent it from fitting onto the piston properly.
- 4. Moisten the silicone ring with a little breast milk. This allows the piston to move more easily in the cylinder and makes pumping easier.
- 5. Push the piston carefully and completely into the cylinder (fig. c).
- 6. Fold the silicone cushion onto the funnel (fig. e).
- 7. Place the funnel on your breast. Make sure that your nipple is exactly in the centre.

8. Use one hand to hold the funnel firmly to your breast and generate pumping movements by moving the cylinder slowly up and down (fig. e) with the other hand.

9. As soon as the milk begins to flow, pump even more slowly. 10. Milk should be expressed from each breast in turn. The amount of time needed to express milk can vary from person to person.

11. When you have expressed enough milk, place the milk in the NUK breast milk container in the refrigerator or freeze it if you need to store it for a longer period (see 5. Storing the Milk).

12. All of the individual components of use must be thoroughly cleaned with water after each use and sterilised before each use.

4. Cleaning and Sterilisation (see fig. d)

To ensure that your baby develops healthy, hygiene is particularly important when handling breast milk. The pump should therefore only be used in a clean and sterilised condition.

4.1. Cleaning

1. Remove the silicone ring and the silicone cushion before cleaning.

2. Clean all of the individual components in the sink using warm water and a liquid soap, e.g. NUK Bottle Cleanser.

Warning!

Cleaning the components in a dishwasher may damage the material.

4.2. Sterilisation

Before sterilising, clean as described in the previous section. Sterilise all of the individual components either:
– in the NUK Steam Steriliser or
– boil for 5 minutes, using sufficient water to avoid damage.

Warning!

Only the NUK Microwave Steam Steriliser should be used for sterilisation in a microwave. Follow the manufacturer's instructions when using any sterilisers or sterilising solution in order to prevent damage to the product.

5. Storing the Milk

If the breast milk is not used immediately, you should pour the breast milk into a suitable container, seal it securely and store it in the refrigerator.

Table with 3 columns: Breast milk, Storage location, Storage periods. Rows: Freshly expressed, At room temperature, Max. 6-8 hours, Refrigerator (not in the door), 72 hours at 4 °C, Refrigerator freezer, 2 weeks, Upright/chest freezer, 6 months at -18 °C

To defrost, place the container in the refrigerator or, if the milk is required quickly, in warm water.

Do not use a microwave or hot water to defrost breast milk as this can destroy valuable nutrients and damage the bottle material. Shake the milk gently before feeding so that the milk and milk fat are properly mixed. Consult your nursing consultant, midwife, doctor or pharmacist for detailed information about long-term storage.

6. Classification

Class I medical device as per EU Directive 93/42/EEC

7. Disposal

If the pump is no longer fit for use, the parts must be disposed of in accordance with local regulations.

8. Warranty

In addition to the warranty obligations of the seller indicated in the sales agreement, the manufacturer shall grant a 24-month warranty for all parts that have not been used as intended and in accordance with the operating instructions starting on the sale date of the pump. The sales receipt shall serve as documentation of the date of sale and the pump type.

Within the warranty period, we pledge to remedy any deficiencies that are based on material or production defects. Wear parts are excluded from the warranty.

Minor deviations from the target condition that are unimportant with regard to the value and usability of the pump shall not activate the warranty. Likewise, the warranty shall not be in effect when the deficiencies of the pump are traceable to transport damage that we did not cause, misuse or insufficient maintenance.

The warranty payment shall be made according to our choice of repair, replacement of parts or replacement of the pump. The payment of warranty shall not result in an extension or a restart of the warranty period. The warranty period for integrated replacement parts shall end with the warranty period for the entire pump.

Extended or other claims, in particular those for replacement outside of the damage caused to the pump shall be excluded unless liability is required by law.

We shall not be liable for transport costs or risks.

If we receive shipment of a pump without any proof of its sale date, we shall treat it as a request for repair. The pump will be repaired only after consulting with the customer.

In case of complaints: save the product number and the lot number.

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